











## Bariatric Bed Ordering Information – Appendix 2A

<b>Bariatric Bed Options</b>		
<b>STEP 1: A bariatric bed may already be available in your facility. First, follow these steps:</b>		
<b>Duke University Hospital</b>	<b>Duke Regional Hospital</b>	<b>Duke Raleigh Hospital</b>
Go to Clinical Engineering > <a href="#">Request for Beds and Equipment Distribution</a>	Call EVS 919-470-4063 to request bariatric bed from 1 <sup>st</sup> floor clean bed hold if one is available	<ul style="list-style-type: none"> <li>• Use bed in North Pavilion, 2<sup>nd</sup> floor, Room 112, if available and fits patient's needs</li> <li>• Take patient label with you to affix to paperwork found on top of bed, complete paperwork, &amp; leave in mounted box outside the door</li> <li>• Place order for rolling walker &amp; bedside commode</li> </ul>
<b>STEP 2: If the appropriate bariatric bed is not available at your facility, determine a bed frame and mattress to order from Agiliti.</b>		
<b>A. Select a bed frame:</b>		
<b>1. BariRehab 3</b> (1000 lbs.; bed width 39 or 48"; scale; bed exit alarm) <ul style="list-style-type: none"> <li>• For patients 5'4" and taller</li> <li>• All BMAT levels</li> <li>• Lowest bed height: 15"</li> <li>• Power drive</li> <li>• Trapeze available</li> </ul> 	<b>2. Expandable Evolution</b> (850 lbs.; bed width 39" or 48"; scale; bed exit alarm) <ul style="list-style-type: none"> <li>• 5'3" and shorter</li> <li>• High fall risk</li> <li>• All BMAT levels</li> <li>• Lowest bed height: 9"</li> <li>• Trapeze available</li> </ul> 	
<b>B. Select a mattress (all surfaces are compatible with the bariatric bed frames):</b>		
<b>1. Big Turn 2 (for ICU)</b> <i>Low air loss; lateral rotation; percussion &amp; vibration; alternation therapy</i> <ul style="list-style-type: none"> <li>• For patients at BMAT levels B, 1, or 2; on ventilator; with tracheotomy; high risk for pressure injury</li> </ul>	<b>2. Mighty Air (for Intermediate/Stepdown)</b> <i>Low air loss; alternation therapy</i> <ul style="list-style-type: none"> <li>• For patients at all BMAT levels; high risk for pressure injury</li> </ul>	<b>3. Foam (Duke Regional only for bariatric surgery)</b> <p>For patients at all BMAT levels; high risk for pressure injury; bariatric by-pass surgery</p>

## Bariatric Equipment and Supply Ordering Information – Appendix 2B

**STEP 3: Determine additional bariatric equipment to care for a patient based on BMAT Mobility Level (refer to BMAT Mobility Based Bariatric Equipment Selection – Appendix 3). If rentals are needed from Agiliti, see the options below.**

### Bariatric Rental Equipment Options

<p><b>Bedside Bari Lift, BMAT Levels 1 &amp; 2</b></p> <ul style="list-style-type: none"> <li>• 1000 lb./455 kg</li> <li>• Scale</li> <li>• For total assistance with mobility</li> </ul> <p><i>CAUTION: Does not lift high enough to clear off BariRehab 3 bed to weigh</i></p> 	<p><b>EZ Way Smart Stand power sit-to-stand lift, BMAT Level 2</b></p> <ul style="list-style-type: none"> <li>• 800 lb./363 kg</li> <li>• Power control</li> <li>• Lift from to stand; Hold in standing</li> </ul> 	<p><b>Shuttle Stretcher-Chair, BMAT Levels 1 &amp; 2</b></p> <p>Shuttle A (650 lb.; Seat Width 25.5") Shuttle B (1000 lb.; Seat Width: 31")</p> <ul style="list-style-type: none"> <li>• Stretcher and chair positions</li> <li>• Power drive available</li> </ul> 	<p><b>Bedside Recliner BMAT Levels 1-4</b></p> <ul style="list-style-type: none"> <li>• 750 lb./340 kg</li> <li>• Seat width 30", depth 21", height: 21.5"</li> </ul> 
<p><b>Bari Drop-Arm Commode, BMAT Levels 1-4</b></p> <ul style="list-style-type: none"> <li>• 600 lbs./273 kg, width 24"</li> <li>• 1000 lbs./386 kg, width 28"</li> </ul> <p>Both with Height: 18" – 22" &amp; adjustable legs</p> 	<p><b>Bari Shower Commode, BMAT Levels 1-4</b></p> <ul style="list-style-type: none"> <li>• 750 lb./340 kg capacity</li> <li>• Width: 26" or 30"; Height: 22"</li> <li>• Removable back, arms, &amp; swing out footrest</li> </ul> 	<p><b>Bari Wheelchair, BMAT Levels 1-4</b></p> <ul style="list-style-type: none"> <li>• 850 lb./386 kg</li> <li>• Seat width: 24" up to 36"</li> <li>• Removable armrests</li> <li>• Elevating, swing away, removable leg rests</li> </ul> 	<p><b>Bari Walker, BMAT Level 4</b></p> <ul style="list-style-type: none"> <li>• 750 lb./340 kg</li> <li>• Width 27"</li> <li>• Grip height 31-40"; adjustable</li> </ul> 

**STEP 4: Order bed frame, mattress, and additional equipment from Agiliti.** You will need the patient's name, room number, weight, and any isolation precautions. Estimated time of delivery is 2 hours plus travel time.

- **Duke University Hospital:** Go to Clinical Engineering > [Request for Beds and Equipment Distribution](#)
- **Duke Regional Hospital:** Call Agiliti at 1-800-814-9389
- **Duke Raleigh Hospital** Weekdays 7 am - 3:30 pm: Call 919-862-5850 After hours & weekends: Call Materials at 919-954-3148

For questions related to the bariatric bed and equipment, call Agiliti at 1-800-814-9389

### Duke Health Bariatric Supplies & SAP #

Bariatric yellow socks	336356	Bariatric Hovermatt-39 inch (1200 lbs.)	338466
Bariatric bedpan	327027	Bariatric loop sling (1000 lbs.) for MaxiSky ceiling lift	347204
Leg lifter	301731	Bariatric repositioning sling (1000 lbs.) for MaxiSky ceiling lift	350838
Bariatric SCD sleeves	320855	Clip bilateral amputee sling (500 lbs.) for MaxiMove lift	189542
Bariatric air offloading boots	355066	Loop bilateral amputee sling (800 lbs.) for MaxiSky ceiling lift	189544
Interdry™ AG	334021	Limb holding sling (154 lbs.) for MaxiSky ceiling lift	350839

No rinse skin cleanser	313131		Bariatric seat cushion (>300 lbs.)	355068
Bariatric morgue kit	17197			