










Bariatric Bed Ordering Information – Appendix 2A

Bariatric Bed Options		
STEP 1: A bariatric bed may already be available in your facility. First, follow these steps:		
Duke University Hospital	Duke Regional Hospital	Duke Raleigh Hospital
Go to Clinical Engineering > Request for Beds and Equipment Distribution	Call EVS 919-470-4063 to request bariatric bed from 1 st floor clean bed hold if one is available	<ul style="list-style-type: none"> • Use bed in North Pavilion, 2nd floor, Room 112, if available and fits patient's needs • Take patient label with you to affix to paperwork found on top of bed, complete paperwork, & leave in mounted box outside the door • Place order for rolling walker & bedside commode
STEP 2: If the appropriate bariatric bed is not available at your facility, determine a bed frame and mattress to order from Agiliti.		
A. Bed frame:		
Umano OOK Snow ALL <ul style="list-style-type: none"> • Patient weight: 70 – 950 pounds (32 – 430 kg) • All BMAT levels • Bed deck height: low 11 inches – high 30 inches • Bed frame width: 35 inches – 47 inches • Bed frame length: 84 inches – 87 inches • Motorized drive for transport • Integrated scale • 3-setting bed exit alarm • Trapeze available 		
		
B. Select a mattress (all surfaces are compatible with the bariatric bed frames):		
1. Big Turn 2 (for ICU) <i>Low air loss; lateral rotation; percussion & vibration; alternation therapy</i> <ul style="list-style-type: none"> • For patients at BMAT levels B, 1, or 2; on ventilator; with tracheotomy; high risk for pressure injury 	2. Mighty Air (for Intermediate/Stepdown) <i>Low air loss; alternation therapy</i> <ul style="list-style-type: none"> • For patients at all BMAT levels; high risk for pressure injury 	3. Foam For patients at low risk for pressure injury, and BMAT level 4, or BMAT level 3 AND NO sensory perception or bed mobility impairment per the Braden subscale, or to meet clinical care needs (e.g., PT, wound care).

Bariatric Equipment and Supply Ordering Information – Appendix 2B

STEP 3: Determine additional bariatric equipment to care for a patient based on BMAT Mobility Level (refer to BMAT Mobility Based Bariatric Equipment Selection – Appendix 3). If rentals are needed from Agiliti, see the options below.

Bariatric Rental Equipment Options

Bedside Bari Lift, BMAT Levels 1 & 2 <ul style="list-style-type: none"> 1000 lb./455 kg Scale For total assistance with mobility 	EZ Way Smart Stand power sit-to-stand lift, BMAT Level 2 <ul style="list-style-type: none"> 800 lb./363 kg Power control Lift to stand; Hold in standing 	Shuttle Stretcher-Chair, BMAT Levels 1 & 2 Shuttle A (650 lb.; Seat Width 25.5") Shuttle B (1000 lb.; Seat Width: 31") <ul style="list-style-type: none"> Stretcher and chair positions Power drive available 	Bedside Recliner BMAT Levels 1-4 <ul style="list-style-type: none"> 750 lb./340 kg Seat width 30", depth 21", height: 21.5" 
Bari Drop-Arm Commode, BMAT Levels 1-4 <ul style="list-style-type: none"> 600 lbs./273 kg, width 24" 1000 lbs./386 kg, width 28" Both with Height: 18" – 22" & adjustable legs 	Bari Shower Commode, BMAT Levels 1-4 <ul style="list-style-type: none"> 750 lb./340 kg capacity Width: 26" or 30"; Height: 22" Removable back, arms, & swing out footrest 	Bari Wheelchair, BMAT Levels 1-4 <ul style="list-style-type: none"> 850 lb./386 kg Seat width: 24" up to 36" Removable armrests Elevating, swing away, removable leg rests 	Bari Walker, BMAT Level 4 <ul style="list-style-type: none"> 750 lb./340 kg Width 27" Grip height 31-40"; adjustable 

STEP 4: Order bed frame, mattress, and additional equipment from Agiliti. You will need the patient's name, room number, weight, and any isolation precautions. Estimated time of delivery is 2 hours plus travel time.

- Duke University Hospital:** Go to Clinical Engineering > [Request for Beds and Equipment Distribution](#)
- Duke Regional Hospital:** Call Agiliti at 1-800-814-9389
- Duke Raleigh Hospital** Weekdays 7 am - 3:30 pm: Call 919-862-5850 After hours & weekends: Call Materials at 919-954-3148

For questions related to the bariatric bed and equipment, call Agiliti at 1-800-814-9389

Duke Health Bariatric Supplies & SAP

Bariatric yellow socks	336356	Bariatric Hovermatt-39 inch (1200 lbs.)	338466
Bariatric bedpan	327027	Bariatric loop sling (1000 lbs.) for MaxiSky ceiling lift	347204
Leg lifter	301731	Bariatric repositioning sling (1000 lbs.) for MaxiSky ceiling lift	350838
Bariatric SCD sleeves	320855	Clip bilateral amputee sling (500 lbs.) for MaxiMove lift	189542
Bariatric air offloading boots	355066	Loop bilateral amputee sling (800 lbs.) for MaxiSky ceiling lift	189544
Interdry™ AG	334021	Limb holding sling (154 lbs.) for MaxiSky ceiling lift	350839
No rinse skin cleanser	313131	Bariatric seat cushion (>300 lbs.)	355068
Bariatric morgue kit	17197		