

Tip of the Month: Got Supplies for MaxiMove?

1. Ensure disposable slings with CLIP attachments are available & accessible; work with Materials Management/Resource Distribution in your area.

- ✓ Store slings for automatic replenish, if able (i.e., Pyxis or Clean Hold)
- ✓ Set a “par level” (set number) for each sling size in Pyxis, *i.e., Size M-3 slings, Size L-6 slings, Size XL-6 slings, Size 2XL- 3 slings*
- ✓ Label sling storage area with sling type, size, & SAP #

Note: Size Small= pediatric



Size	SAP #	Name
S Red	346206	Sling, Clip Lift Arjo S-Peds
M Yellow	313289	Sling, Clip Lift Arjo M
L Green	313290	Sling, Clip Lift Arjo L
XL Blue	313291	Sling, Clip Lift Arjo XL
XXL Dark Red	335345	Sling, Clip Lift Arjo 2XL



Sling storage labels

2. Ensure each MaxiMove has TWO batteries- one for the lift and one spare.

- ✓ Alternate batteries and charge once every two weeks or when low
- ✓ Label each battery “A” and “B”
- ✓ If a battery is dead or does not stay charged, submit work request to Clinical Engineering to repair or purchase new battery or charger

