

Tip of the Month: Got Supplies for SaraPlus?

1. Ensure the wipeable and reusable belts are available & accessible.

- ✓ Each lift has belts in THREE sizes: M, L, and XL
- ✓ Older version = green New version = gray/blue



- ✓ Mount hooks on wall to store; clip belt together before hanging so it won't slip out
- ✓ Replace missing or torn belts by ordering from Procurement



Size	SAP #	Name
M Yellow	334065	SaraPlus wipe-down MED
L Green	334066	SaraPlus wipe-down LARGE
XL Blue	334067	SaraPlus wipe-down XL

Order replacement belts by submitting a requisition to Procurement in 3 ways: 1) electronically thru SAP; 2) scan & email to procurement@duke.edu; or 3) fax 919-684-4344

2. Ensure each SaraPlus has TWO batteries- one for the lift and one spare.

- ✓ Alternate batteries and charge once every two weeks or when low
- ✓ Label each battery "A" and "B"
- ✓ If battery is dead or does not stay charged, submit work request to Clinical Engineering to repair or purchase new battery or charger

