

Tip of the Month:

Troubleshoot MaxiSky 600 lb.* Lift



Problem:	Action:
<p>MaxiSky does not work when buttons on hand control are pressed</p>	<p>Check if red emergency stop cord is pulled out; to push in, use hanger bar or ask for assistance from Engineering and Operations.</p> <div style="display: flex; justify-content: space-around;"> <div data-bbox="843 585 1146 685"> <p>Emergency stop pulled out</p> </div> <div data-bbox="1159 459 1533 708"> </div> <div data-bbox="1635 462 2018 743"> </div> </div>
<p>GREEN or RED light is on and nothing shows on the display</p>	<p>MaxiSky may be in "sleep mode". Press a button on the hand control to see if something appears on the display. If the lift is still not working, enter Clinical Engineering work order.</p>
<p>Lift emits beeping sound, RED light is on, or empty battery icon appears</p> <div data-bbox="389 1068 606 1286"> </div>	<ul style="list-style-type: none"> • Enter Clinical Engineering work order to replace the battery • Place "Out of Order" label on hand control • Communicate to manager and colleagues



**Only for continuous charge track where there is no charging station including MaxiSky 600 lb. lifts at DRAH, DMP, and DCT.*