

HillRom Progressa™ Bed with IntelliDrive®

Occasionally, the HillRom Progressa™ bed with the IntelliDrive® Transport System may need to have the drive wheel disengaged during transport to safely move the equipment without injury to staff or patients.

If there is no power for the IntelliDrive®, disengage the electronic brake prior to manual push.

Disengaging the IntelliDrive® Electronic Brake

Unplugging the bed, and putting the bed in steer mode will automatically deploy the drive wheel, but will not power the IntelliDrive® Transport System.

To disable the IntelliDrive® electronic brake (in case of malfunction or loss of power):

1. Access the IntelliDrive® wheel box underneath the bed from the patient's right side if looking from the head of the bed (near the CPR lever).
2. Then, lift toggle switch to disengage the IntelliDrive® and allow manual push.



Sequence Label & Battery Life Indicator

Be sure to check the battery indicator on the sequence label located between the transport handles, especially prior to transport.



Call HillRom for on-site for assistance at 919-681-0343.